


BONEY PELVIS WO – SRS MRI PROTOCOL

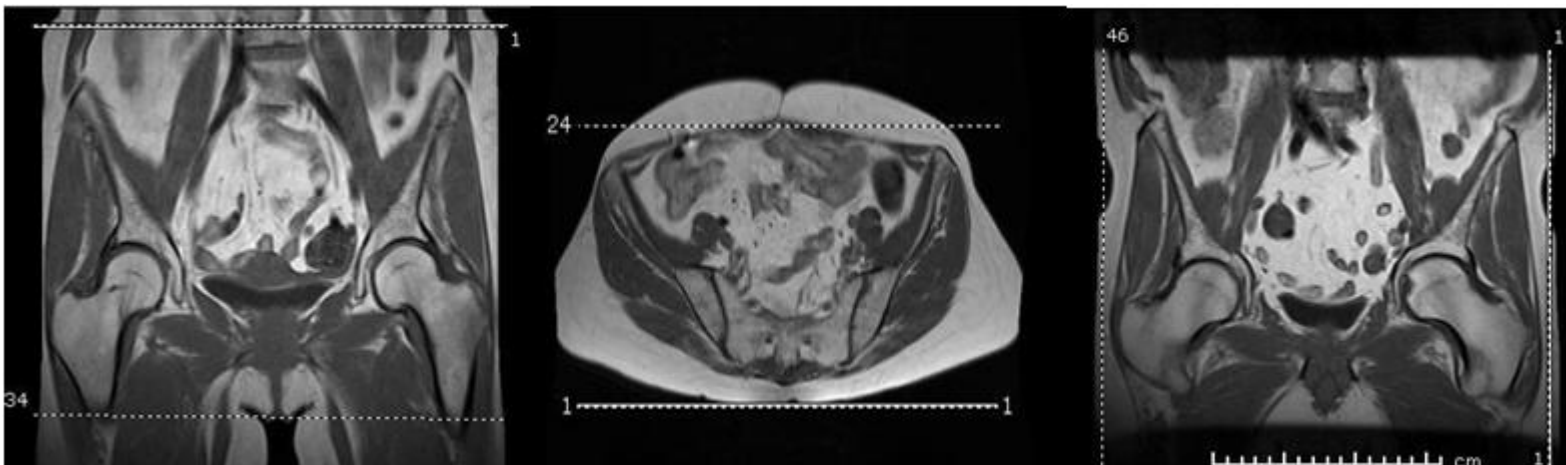

Published: 1/2026			Rev: 1			Approval: Dr Tompkins					Magnet/Vendor: Siemens 3.0T XQ				
Seq	Pulse	Dim	Ori	Scan Dir	Phase Enc	FOV (mm)	Slc (mm)	Gap (mm)	TR	TE	Fat Sat	Freq	Phase	Notes	
LOCALIZER	VIBE	2D	Sag	R>>L	A>>P	420	6	1.2	7.8	3.69	None	256	205		
COR T1	tse	2D	Cor	A>>P	R>>L	350	4	1.0	797	9.8	None	288	202		
COR STIR	tir_rr	2D	Cor	A>>P	R>>L	340	4	1.0	7400	88	IR	304	213	TI 205 ms	
AX T1	tse	2D	Axial	S>>I	A>>P	360	4	1.0	706	20	None	304	213		
AX STIR	tir	2D	Axial	S>>I	A>>P	360	4	1.0	4500	29	IR	320	224	TI 220 ms	
SAG T1	tse	2D	Sag	R>>L	H>>F	340	5	0.5	785	10	None	320	224		
SAG STIR	tir	2D	Sag	R>>L	H>>F	340	5	0.5	5200	29	IR	304	213	TI 220 ms	