


L-SPINE WWO

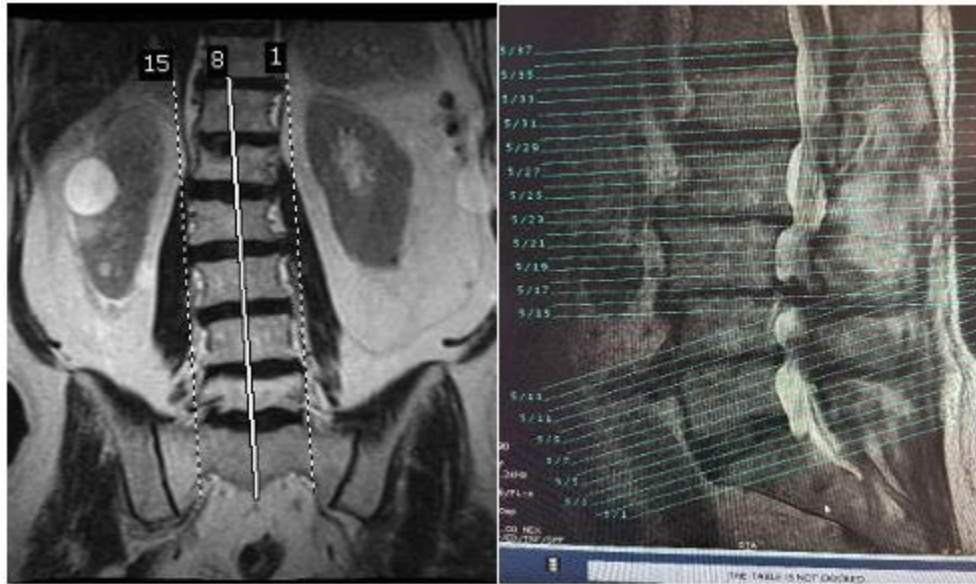

Magnet Vendor/Field: SIEMENS MAGNETOM
3.0T

Approved by: Dr Matos

Published/Rev: 7.2026 / 3

Seq	Pulse	Dim	Ori	Scan Dir	Phase Enc	FOV (mm)	Slc (mm)	Gap (mm)	TR (ms)	TE (ms)	Fat Sat	Freq	Phase	Notes
Localizer	Localizer	2D	Sag	R>>L	R>>L	280	6.0	9.0	7.5	3.36	None	256	205	
SAGITTAL T2	TSE	2D	Sag	R>>L	H>>F	230	3.0	0.5	3000.0	100.00	None	384	269	
SAGITTAL T1	TSE	2D	Sag	R>>L	H>>F	230	3.0	0.5	500.0	9.50	None	384	269	
SAGITTAL IR	IR	2D	Sag	R>>L	A>>P	230	3.0	0.5	3650.0	62.00	IR	384	250	TI 180 ms
AXIAL T2	TSE	2D	Ax	S>>I	A>>P	180	3.0	0.5	3500.0	95.00	None	384	300	
SAGITTAL DIFFUSION	Diff	2D	Sag	R>>L	A>>P	230	3.0	0.5	3060.0	58, 93	FS (Strong)	160	160	ONLY RUN FOR MASS OR INFECTION b-values 0, 800

ADC Mapping	ADC Map													create an ACD Map and send to PACS
AXIAL T1 PRE	TSE	2D	Ax	S>>I	A>>P	180	3.0	0.5	700.0	12.00	FS (Strong)	384	300	
SAGITTAL T1 POST	TSE	2D	Sag	R>>L	H>>F	230	3.0	0.5	733.0	7.50	FS (Strong)	384	269	
AXIAL T1 POST	TSE	2D	Ax	S>>I	A>>P	180	3.0	0.5	700.0	12.00	FS (Strong)	384	300	