


LIVER WWO

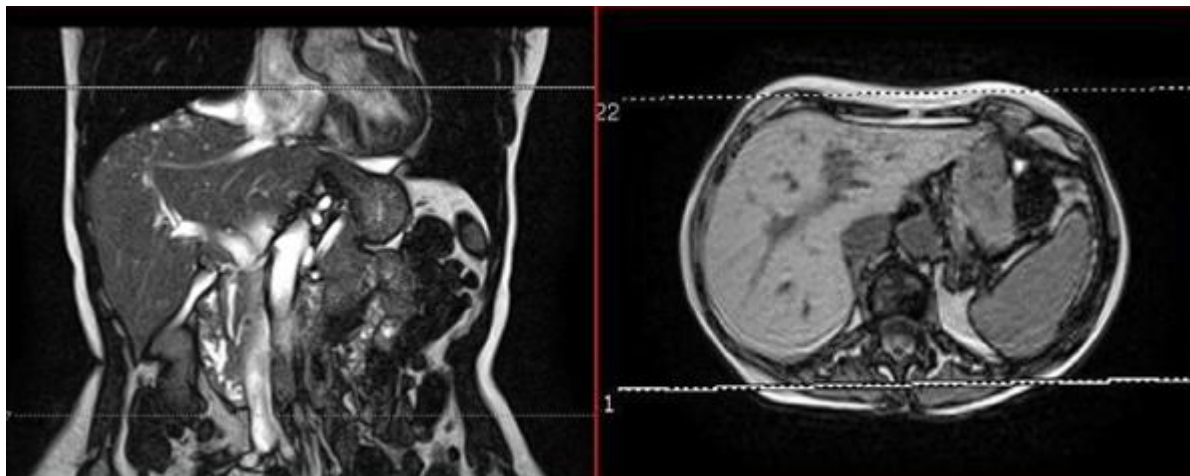

Published: 3/2026				Rev: 2				Approval: Dr Quiroz				Magnet/Vendor: SIEMENS MAGNETOM 3.0T XQ Numaris/X VA60A-0CN3		
Seq	Pulse	Dim	Ori	Scan Dir	Phase Enc	FOV (mm)	Slc (mm)	Gap (mm)	TR	TE	Fat Sat	Freq	Phase	Notes
Localizer	VIBE	2D	Sag	R>>L	A>>P	400	6.0	1.2	9.5	4.36	None	256	180	
CORONAL T2 HASTE MBH	HASTE	2D	Cor	A>>P	R>>L	380	6.0	1.2	1300.0	87.00	None	320	320	
AXIAL T2 FS BLADE NAV	ftseBN	2D	Ax	S>>I	R>>L	380	5.0	1.5	3000.0	84.00	SPAIR (Weak)	256		
AX T2 HASTE MBH	HASTE	2D	Ax	S>>I	A>>P	380	5.0	1.0	1500.0	95.00	None	320	260	
AXIAL T1 VIBE DIXON	VIBE	3D	Ax	S>>I	A>>P	380	3.0	0.6	4.02	1.29 / 2.52	Dixon	320	240	Send In/Out/Fat/Water to PACS
AXIAL T1 VIBE PRE	VIBE	3D	Ax	S>>I	A>>P	380	3.0	0.6	3.6	1.38	FS	320	269	
CARE BOLUS SAG FLUORO	VIBE	2D	Sag	R>>L	H>>F	360	34.0	6.8	32.13	1.68	None	320	236	
AXIAL T1 VIBE POST 1	VIBE	3D	Ax	S>>I	A>>P	380	3.0	0.6	3.6	1.38	FS	320	269	create a subtraction (post-pre)
AXIAL T1 VIBE POST 2	VIBE	3D	Ax	S>>I	A>>P	380	3.0	0.6	3.6	1.38	FS	320	269	create a subtraction (post-pre)

AXIAL T1 VIBE POST 3	VIBE	3D	Ax	S>>I	A>>P	380	3.0	0.6	3.6	1.38	FS	320	269	create a subtraction (post-pre)
AXIAL DIFFUSION	epse	2D	Ax	S>>I	A>>P	380	5.0	1.5	8100.0	55.00	SPAIR (Strong)	134	134	<i>b50 / b800</i>
ADC Mapping	derived													<i>Create ADC map and send to PACS</i>
AXIAL T1 VIBE POST 4	VIBE	3D	Ax	S>>I	A>>P	380	3.0	0.6	3.6	1.38	FS	320	269	create a subtraction (post-pre)
CORONAL T1 VIBE POST 5 MIN DELAY	VIBE	3D	Cor	A>>P	R>>L	420	3.5	0.7	3.66	1.25	FS	320	224	

COMBINE ALL INTO A SINGE SUBTRACTED SERIES TO SEND TO PACS