



MALE PELVIS SEED PROTOCOL

Magnet Vendor/Field	Approved By	Published/Rev
Siemens MAGNETOM 3.0T XQ	Dr Quiroz	6.2025 / 1

PROTOCOL SUMMARY

Protocol	Contrast	Estimated Scan Time
MALE PELVIS SEED PROTOCOL	No	~15-25 min

TECHNOLOGIST WORKFLOW

#	Step	Seq	Pulse	Dim	Ori	Scan Dir	Phase Enc	FOV (mm)	Slc (mm)	Gap (mm)	TR (ms)	TE (ms)	Fat Sat	Freq	Phase	Notes
1	♥ Setup	LOCALIZER	VIBE	2D	Sag	R>>L	A>>P	420	6.0	1.2	7.8	3.69	None	256	205	
2	■ T2	CORONAL T2 FS	TSE	2D	Cor	A>>P	R>>L	400	4.0	1.0	3000	61	FS (Weak)	320	224	
3	■ T2	SAGITTAL T2 FS	TSE	2D	Sag	R>>L	A>>P	220	4.0	1.0	4770	125	FS (Weak)	384	269	
4	■ T1	AXIAL T1	TSE	2D	Ax	S>>I	R>>L	280	3.0	0.0	600	11	None	384	288	
5	■ T2	AXIAL T2 FS	TSE	2D	Ax	S>>I	R>>L	280	3.0	0.0	4340	110	FS (Weak)	320	320	
6	■ 3D	AXIAL T2 3D SPACE	SPACE	3D	Ax	S>>I	A>>P	400	2.0	0.2	1600	218	None	512	486	