



PELVIC HERNIA WO

Magnet Vendor/Field	Approved By	Published/Rev
Siemens MAGNETOM 3.0T XQ	Dr Quiroz	6.2026 / 1

Protocol	Contrast	Estimated Scan Time
PELVIC HERNIA WO	No	~10–15 min

#	Step	Seq	Pulse	Dim	Ori	Scan Dir	Phase Enc	FOV (mm)	Slc (mm)	Gap (mm)	TR (ms)	TE (ms)	Fat Sat	Freq	Phase	Notes
1	♥ Setup	LOCALIZER	VIBE	2D	Sag	R>>L	A>>P	420	6.0	1.2	7.8	3.69	None	256	205	
2	■ T2	CORONAL T2 HASTE	HASTE	2D	Cor	A>>P	R>>L	400	7.0	1.4	2000	82	None	320	256	
3	■ T2	AXIAL T2 FS BH	TSE	2D	Ax	S>>I	A>>P	300	4.0	1.0	2000	109	FS (Weak)	320	288	Breath hold
4	■ T1	AXIAL T1	TSE	2D	Ax	S>>I	A>>P	300	4.0	1.0	750	20	FS (Strong)	576	403	
5	■ T2	SAGITTAL T2 SMALL FOV (PENIS)	TSE	2D	Sag	R>>L	A>>P	180	4.0	0.5	3500	106	None	320	256	Focused penile anatomy
6	■ T2	AXIAL T2 SMALL FOV (PENIS)	TSE	2D	Ax	S>>I	A>>P	180	4.0	0.5	6990	106	None	320	256	Focused penile anatomy

TECHNOLOGIST POST-PROCESSING

- ☐ Verify all series sent to PACS
- ☐ Review image quality before patient leaves



☐ Confirm small FOV coverage of region of interest

ACTION ITEMS

☐ Verify dedicated small FOV penile images adequately cover suspected hernia region